
# OHROPAX® Kids

EN 352-2:2020, EU 2016/425

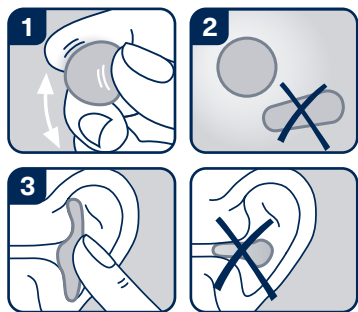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## EN Reusable silicone earplugs against harmful noise

**Instructions for use (see picture):** 1. Knead the silicone plug thoroughly until the material becomes pleasantly soft. 2. Shape into a ball (no elongated form!). 3. Seal the ear canal tightly by gently pressing the ball. **Do not insert into the ear canal! Use entire earplug – do not split!**

**Removal:** Use your fingers to remove the plug in one piece. Do not use any auxiliary means! The plug can be loosened by slightly pushing the back of the ear.

**Further information:** • Insert the earplugs as described and check that they fit correctly. • This product is intended only for people with smaller ear canals. • Ensure hygienic use. Your hands, ears and earplugs should be clean and dry. • The earplugs should be regularly inspected for usability as well as for the necessity of cleaning. • If they still adhere to your ear and are clean and dry, they can be reused. • Cleaning: Light impurities can be carefully wiped off using a cloth and warm water. Wait until completely air-dried. • Wear earplugs at all times in noisy environments. • For water protection: Do not use when diving at depths greater than 1 m. Highly active movements in water, such as jumping or freestyle swimming, may loosen the plugs. • Ask your physician before using earplugs if you have an ear infection, an eardrum injury or a grommet inside the ear. • Failure to comply with these instructions may significantly impair the protective effect of the product. • This product may be adversely affected by certain chemical substances (e.g. hand cream). Further information can be sought from the manufacturer. • Keep away from small children (danger of swallowing). • Can be used by children aged six and older; use only under adult supervision. • Ensure that the earplug is sufficiently large to prevent accidental insertion into the ear canal. • If removal with fingers becomes impossible, seek medical advice.

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- A** CHEMOMEDICA GmbH, 1010 Wien, www.chemomedica.at
- P** Uriach Lusa, S.A., 1495-061 Miraflores, www.uriach.com
**OHROHP GmbH**, Am Kapfengarten 17